



## FOOD PREP

### 1 PREPARE FOOD

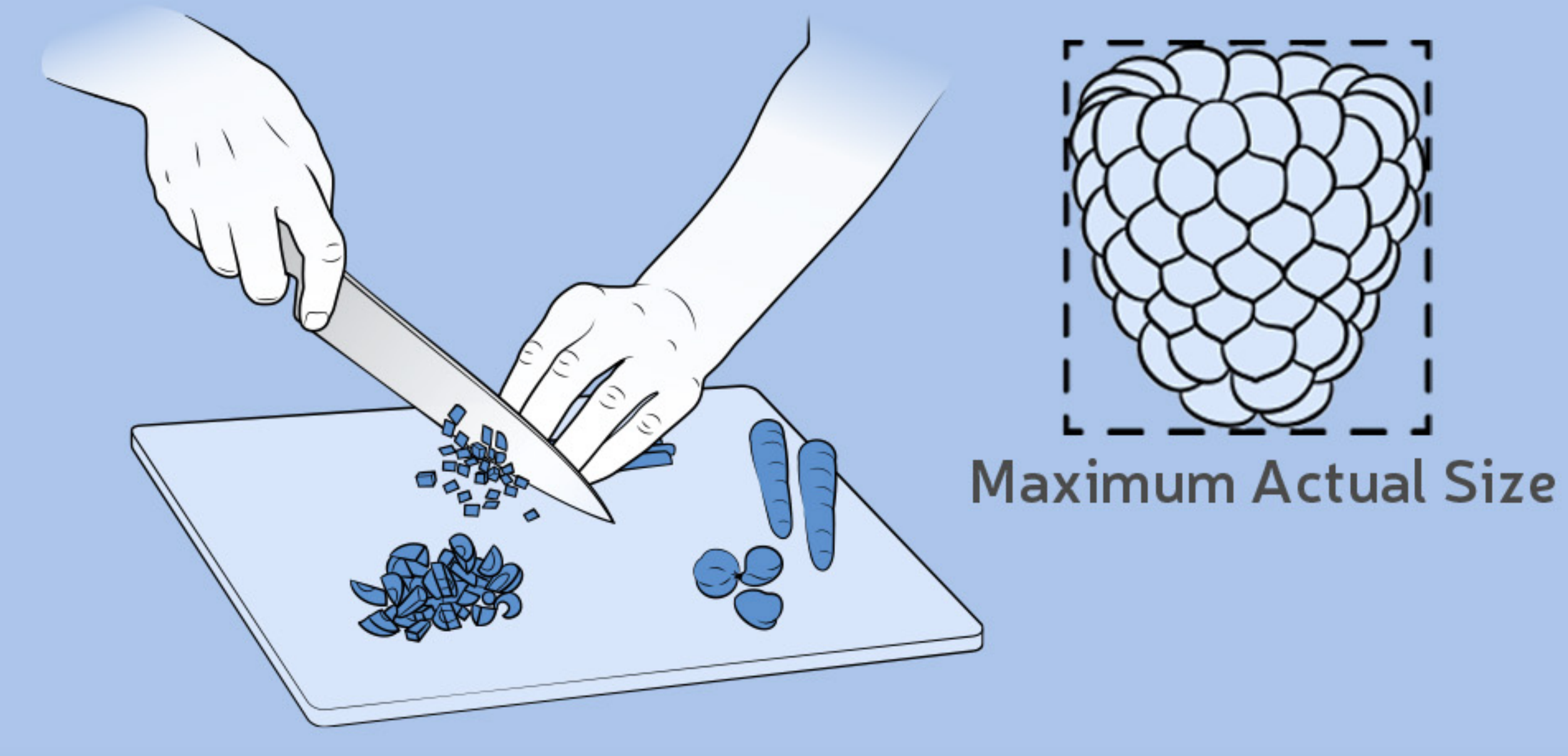


**Solids:** Prepare pieces no larger than the size of a raspberry.  
**Semi-solids:** Add extra liquid to sticky foods. ie. oatmeal, mashed potatoes, mac and cheese, etc.

### 2 FILL BOWLS

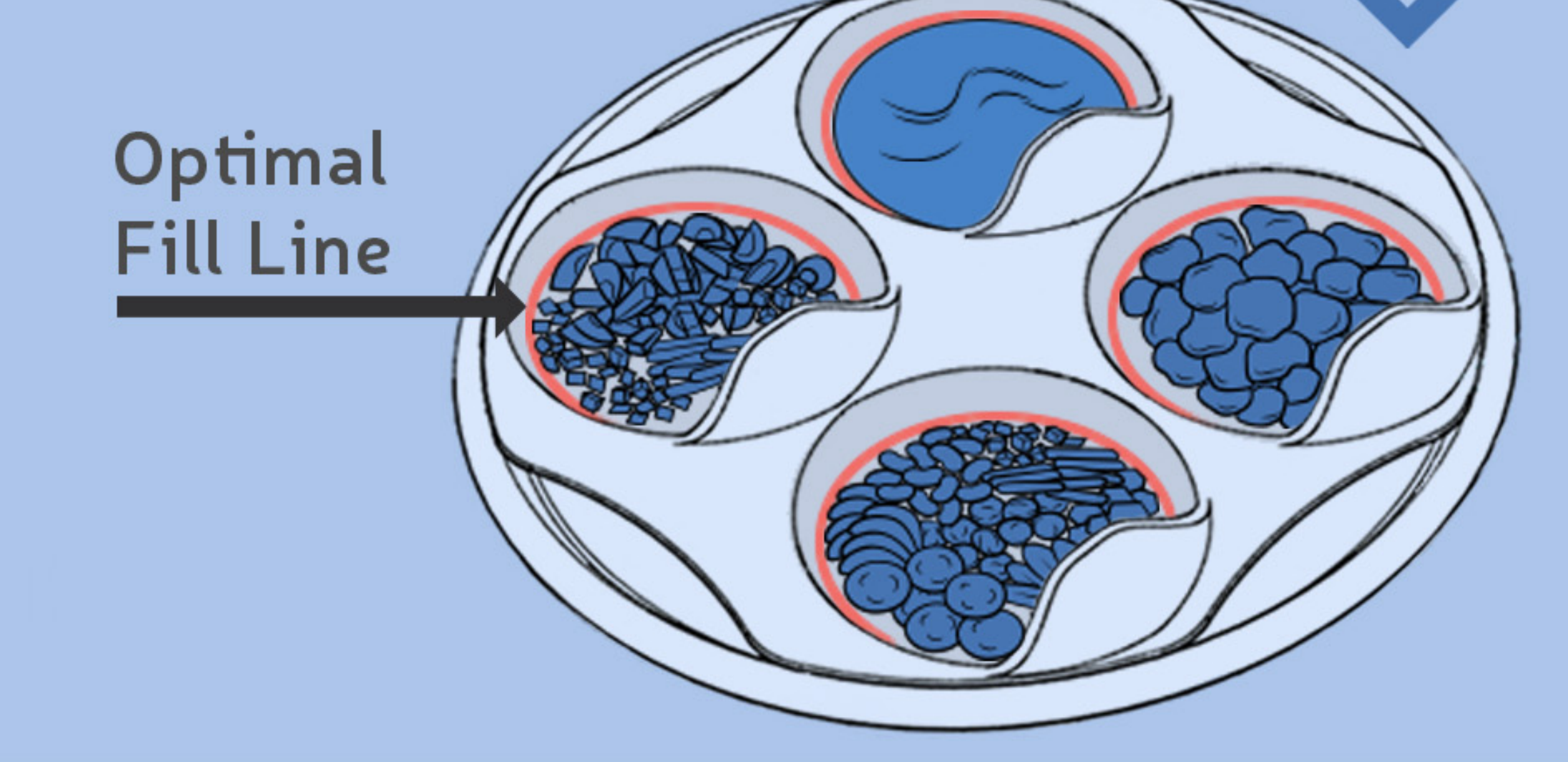

Fill the bowls up to, but not exceeding, the Optimal Fill Line.


### 3 CHECK FILL LEVEL

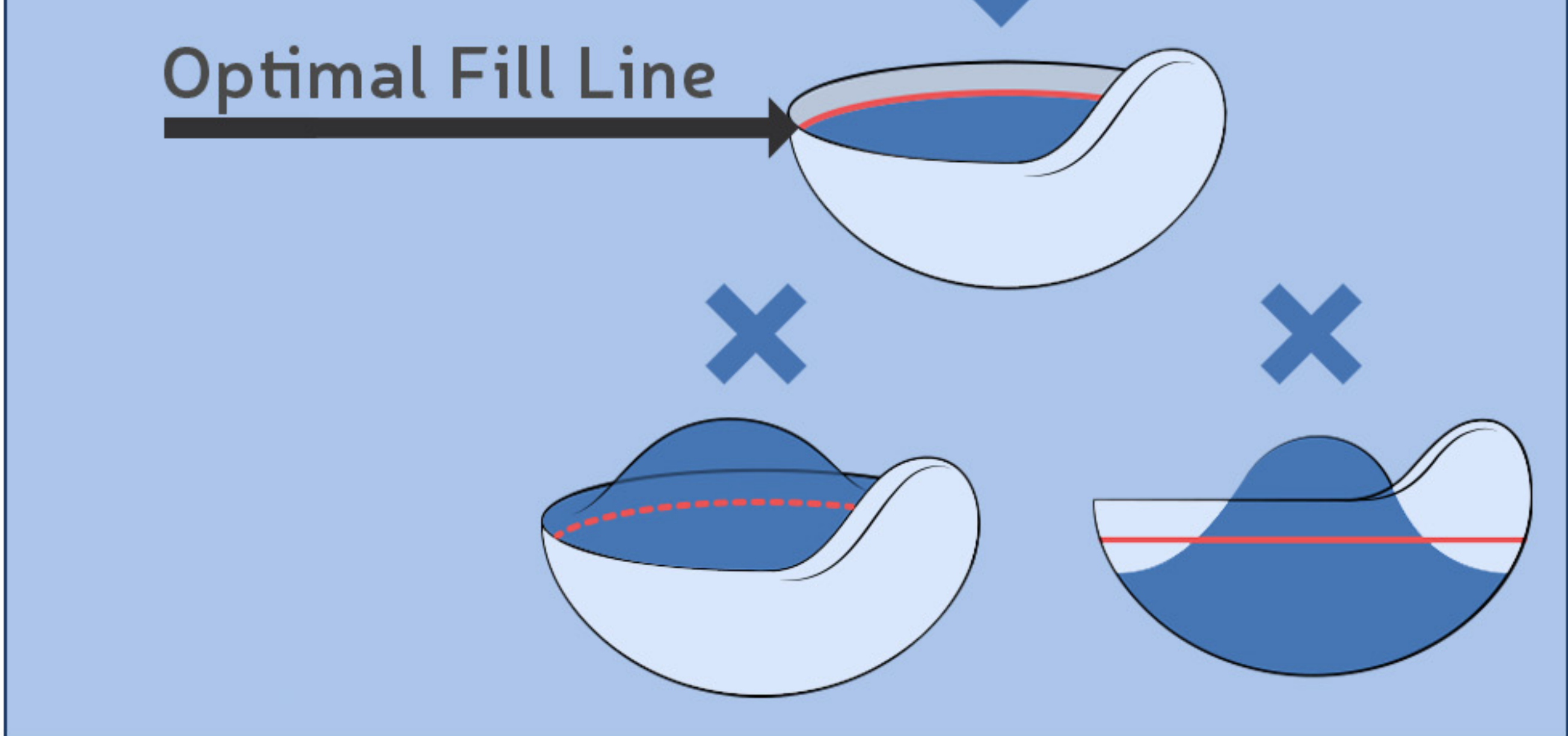

Ensure food is not above or below the Optimal Fill Line. Underfilling is not recommended.


## SETUP

### 4 START POSITION

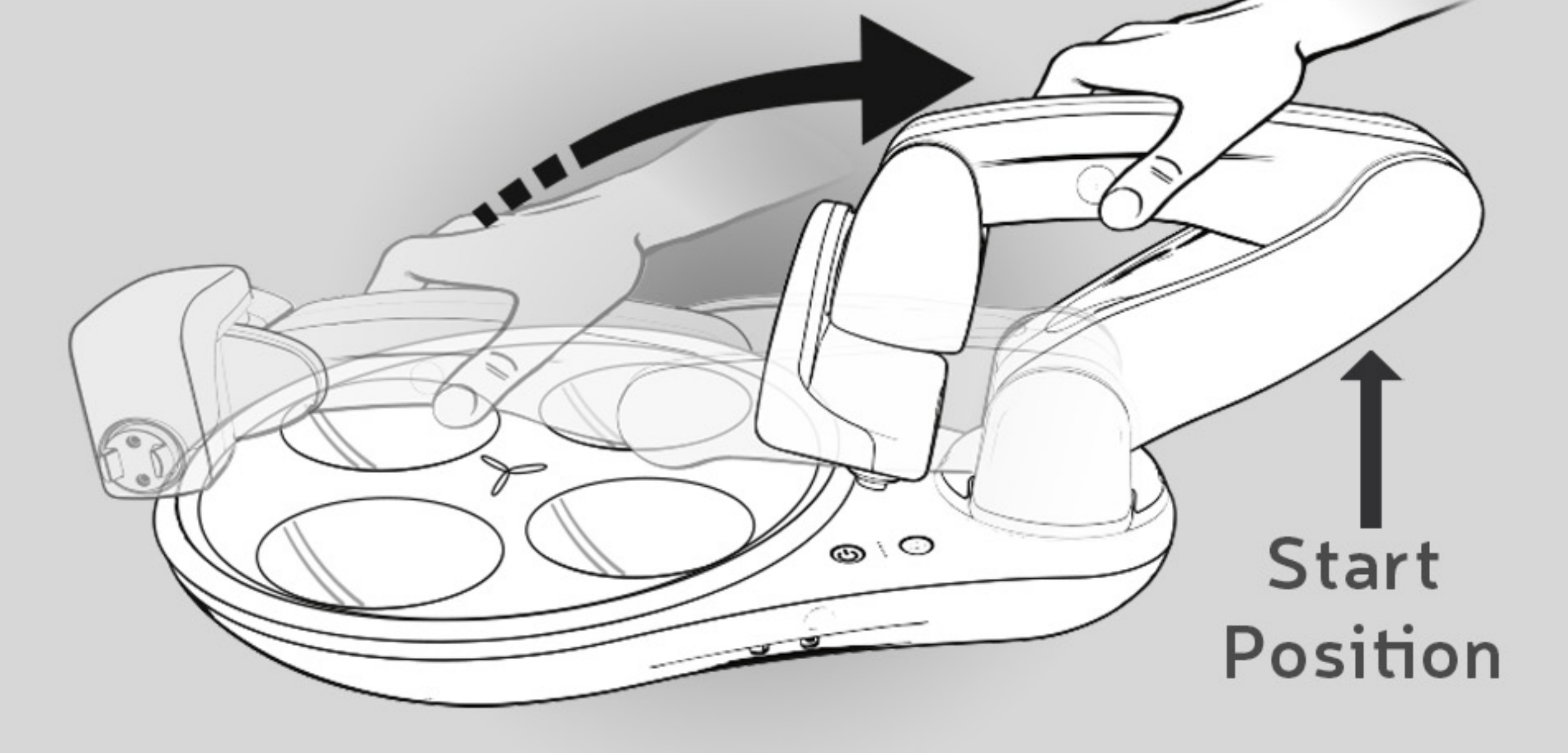


Position the device on a flat surface. Slowly move the arm to the bent Start Position, as shown. The feeling of resistance is normal. Obi will not turn on unless it's in the Start Position.

### 5 INSERT PLATE, PLACEMAT & SWITCHES

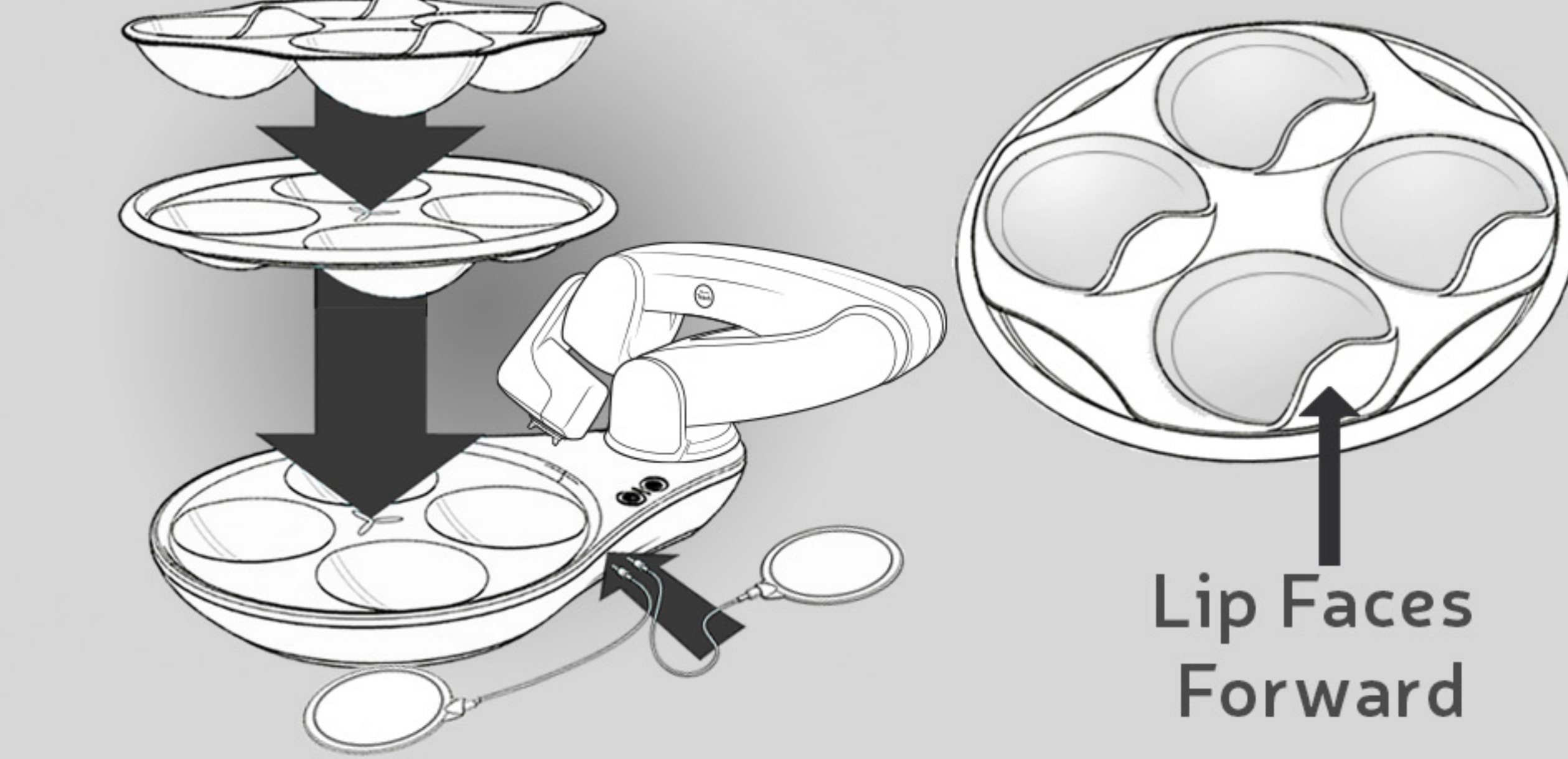

Clean plate and utensil before first use. Insert the black placemat, then place the plate on top. Insert switches.


### 6 ATTACH UTENSIL

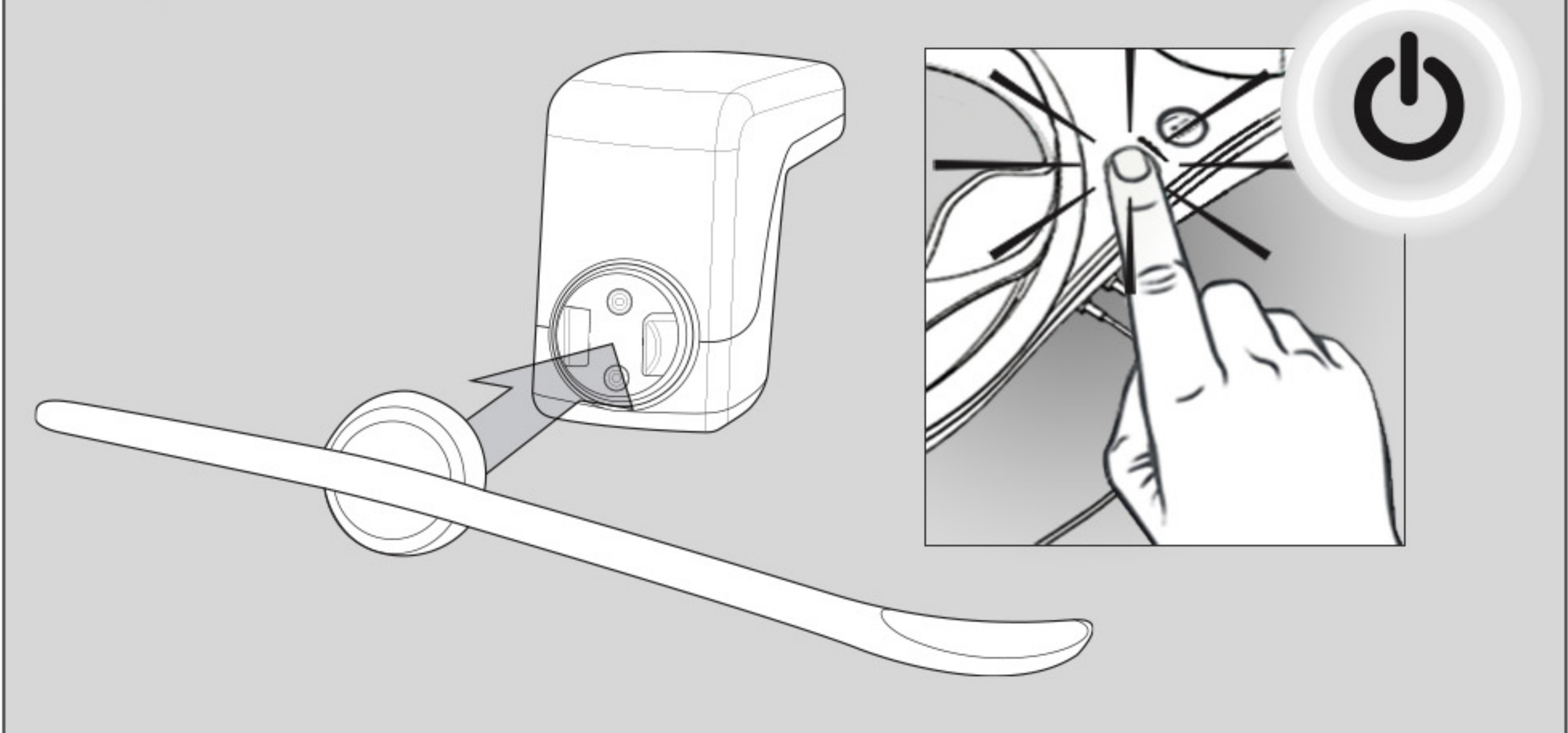

Attach the desired utensil, then press the power button. Obi will turn on and bow before the meal.


## TEACH MODE

### 7 ENTER TEACH MODE

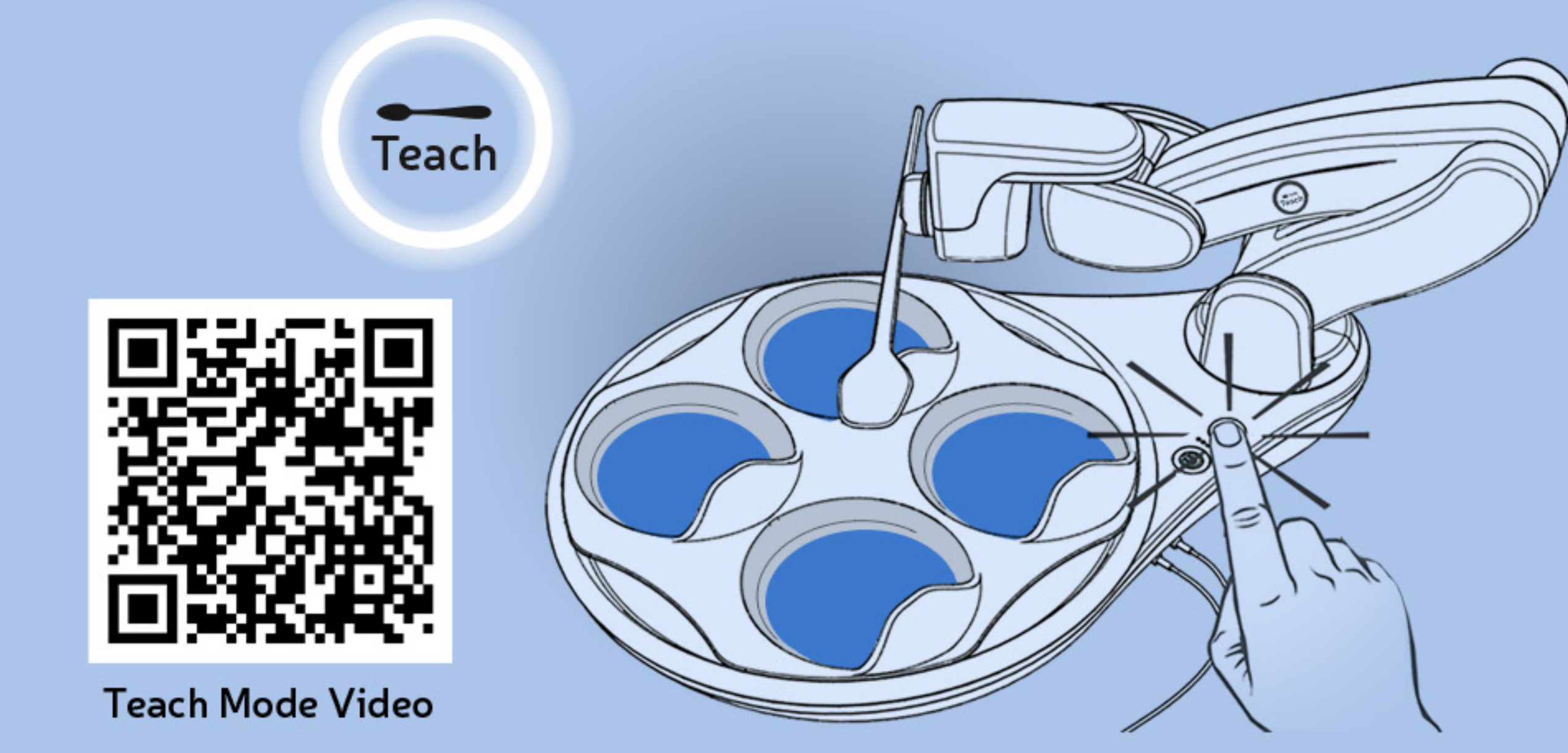

Position the user to be sufficiently upright and stationary. Press the teach button.


### 8 DETERMINE LOCATION

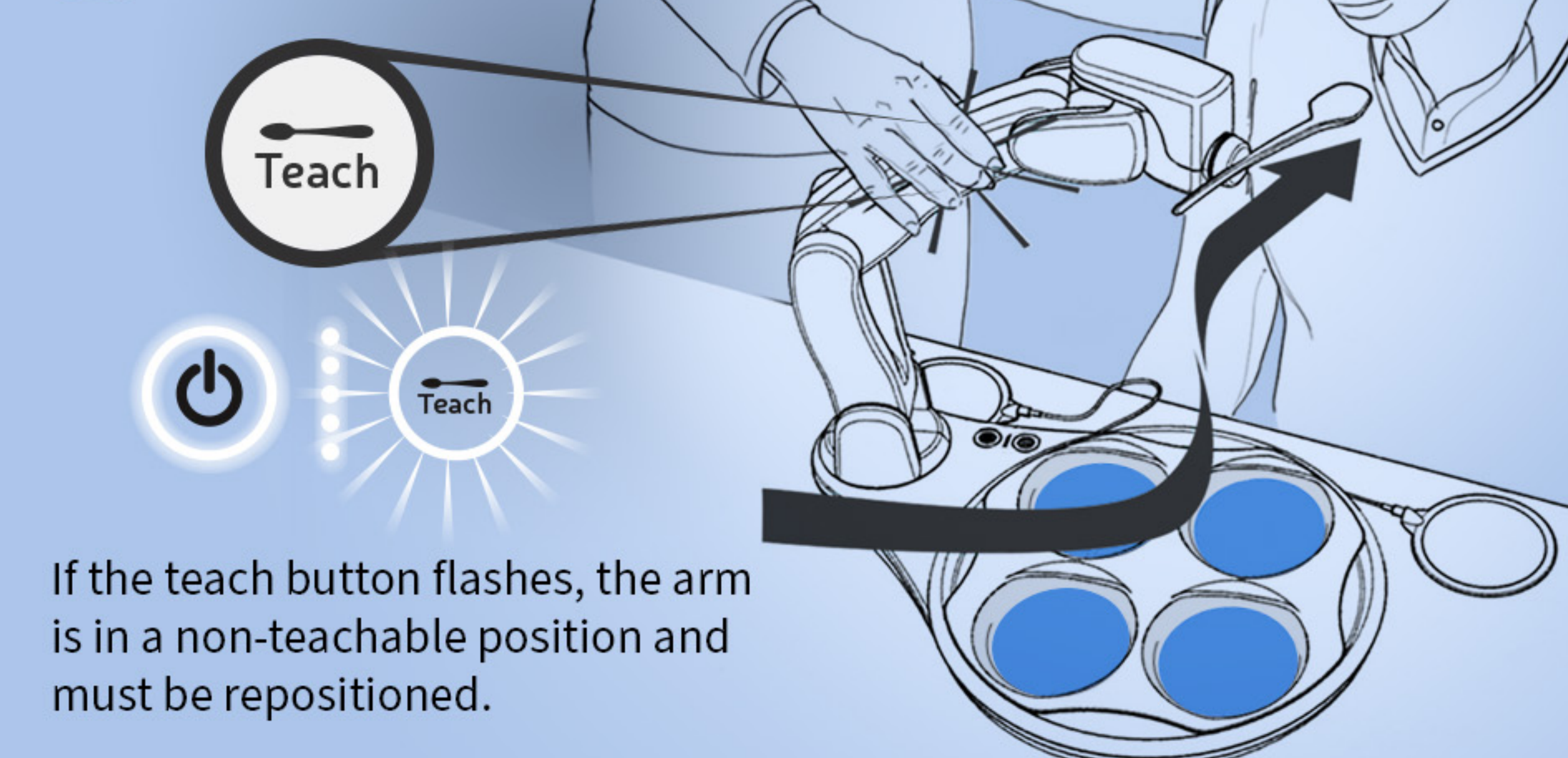

If the teach button flashes, the arm is in a non-teachable position and must be repositioned.

Grasp arm with the thumb covering the teach icon. Support the weight of the arm by lifting up. Move arm to desired delivery location outside of the users mouth. Once at the desired location, let go of arm. Obi will hold the position.


### 9 EXIT TEACH MODE

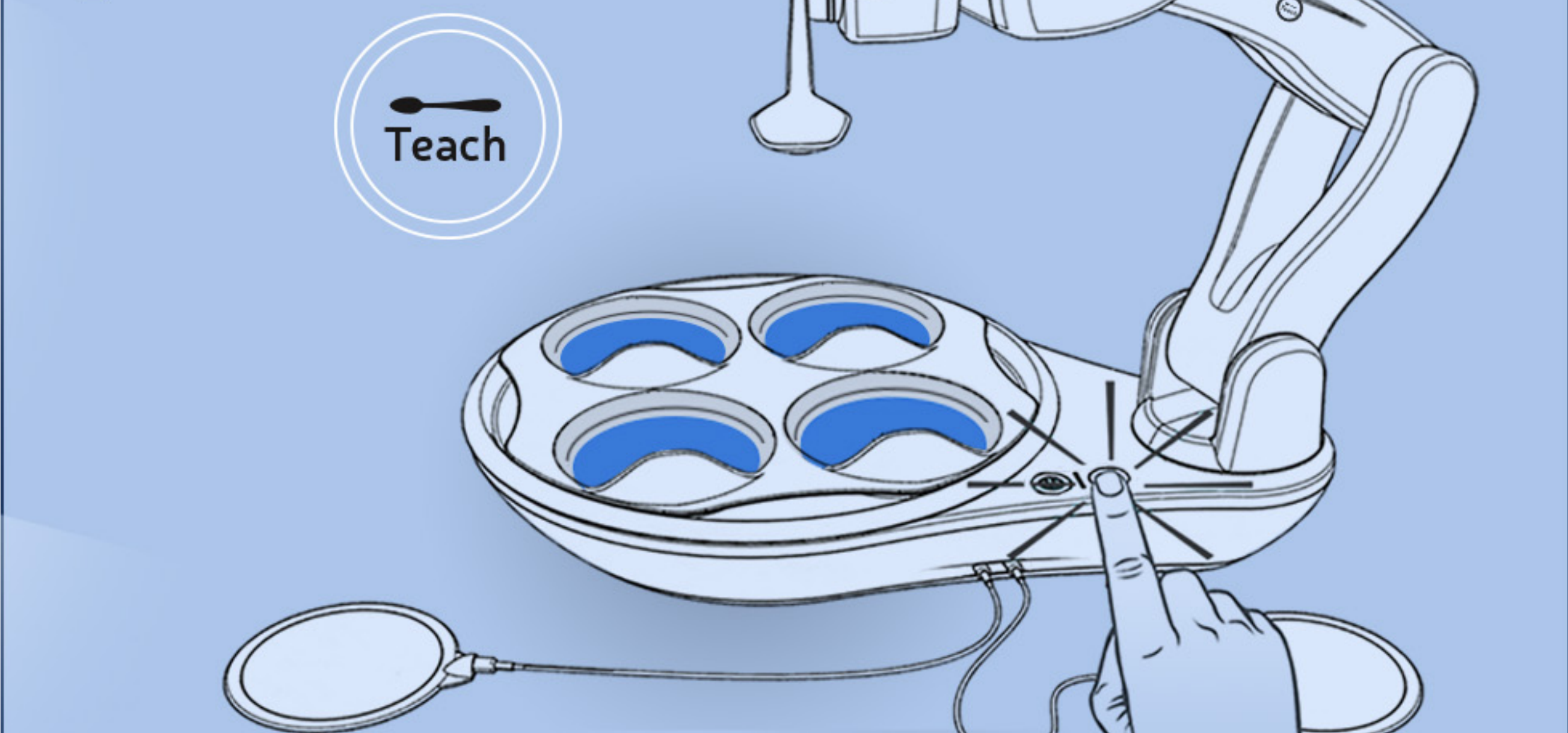


Press the teach button again to store the position. Obi will exit teach mode and move to the ready position above the bowl.  
 ⚠️ WARNING: CHOKING HAZARD. Do not teach a delivery position inside of the mouth.

# Quick Start Guide


## EATING

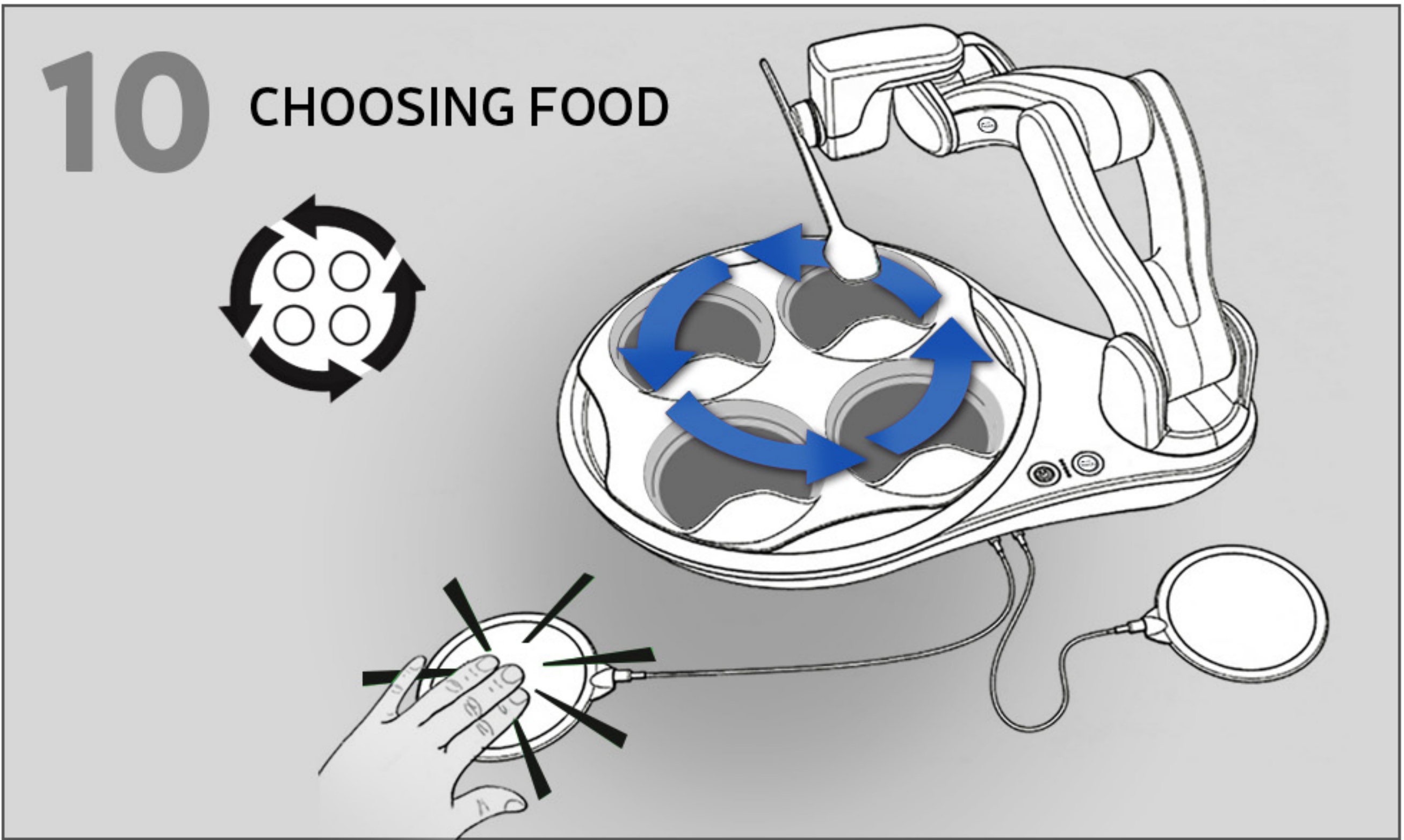

Press the choose switch to move the utensil counter clockwise.

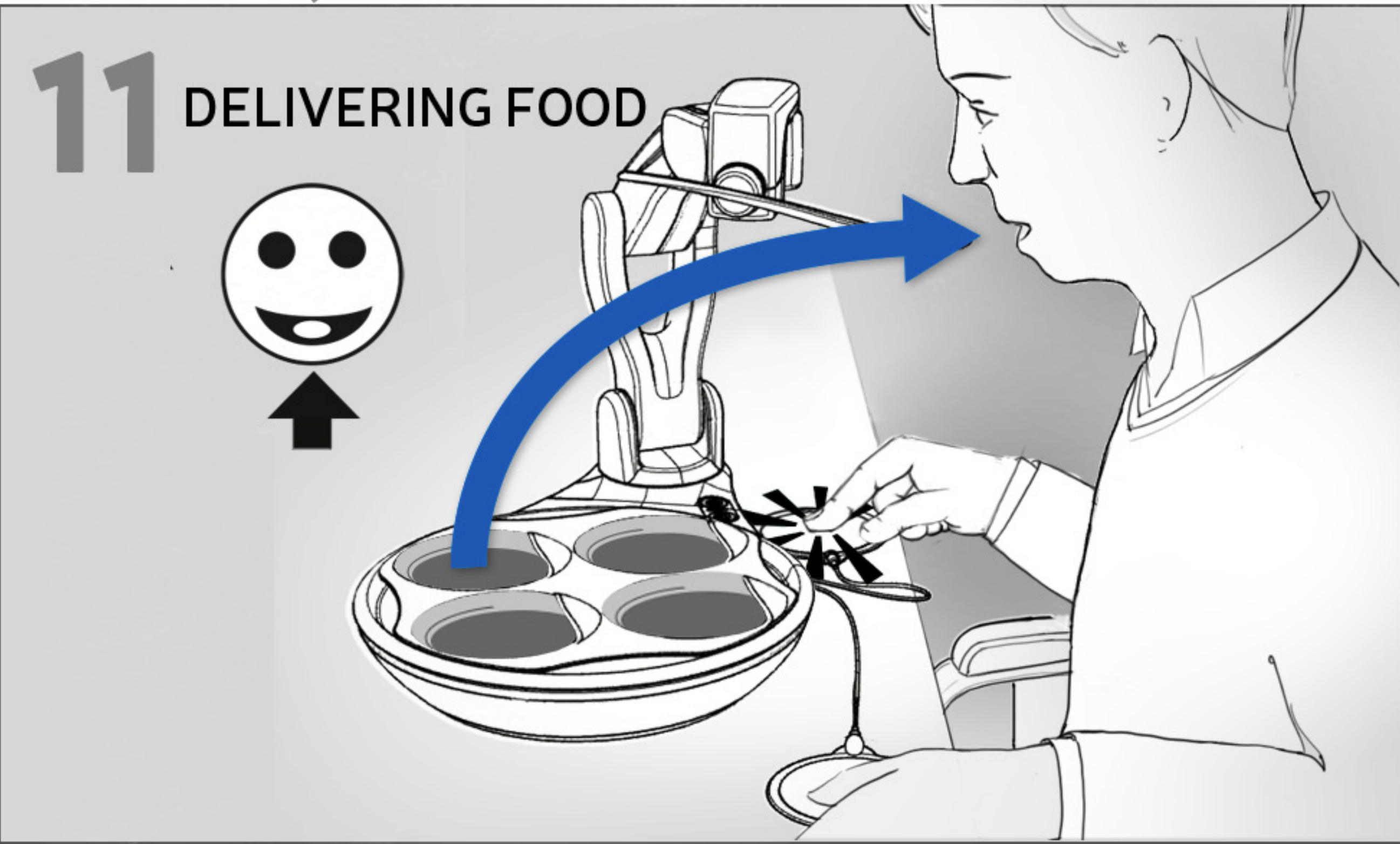


Press the deliver switch to scoop and transport food to the delivery location.

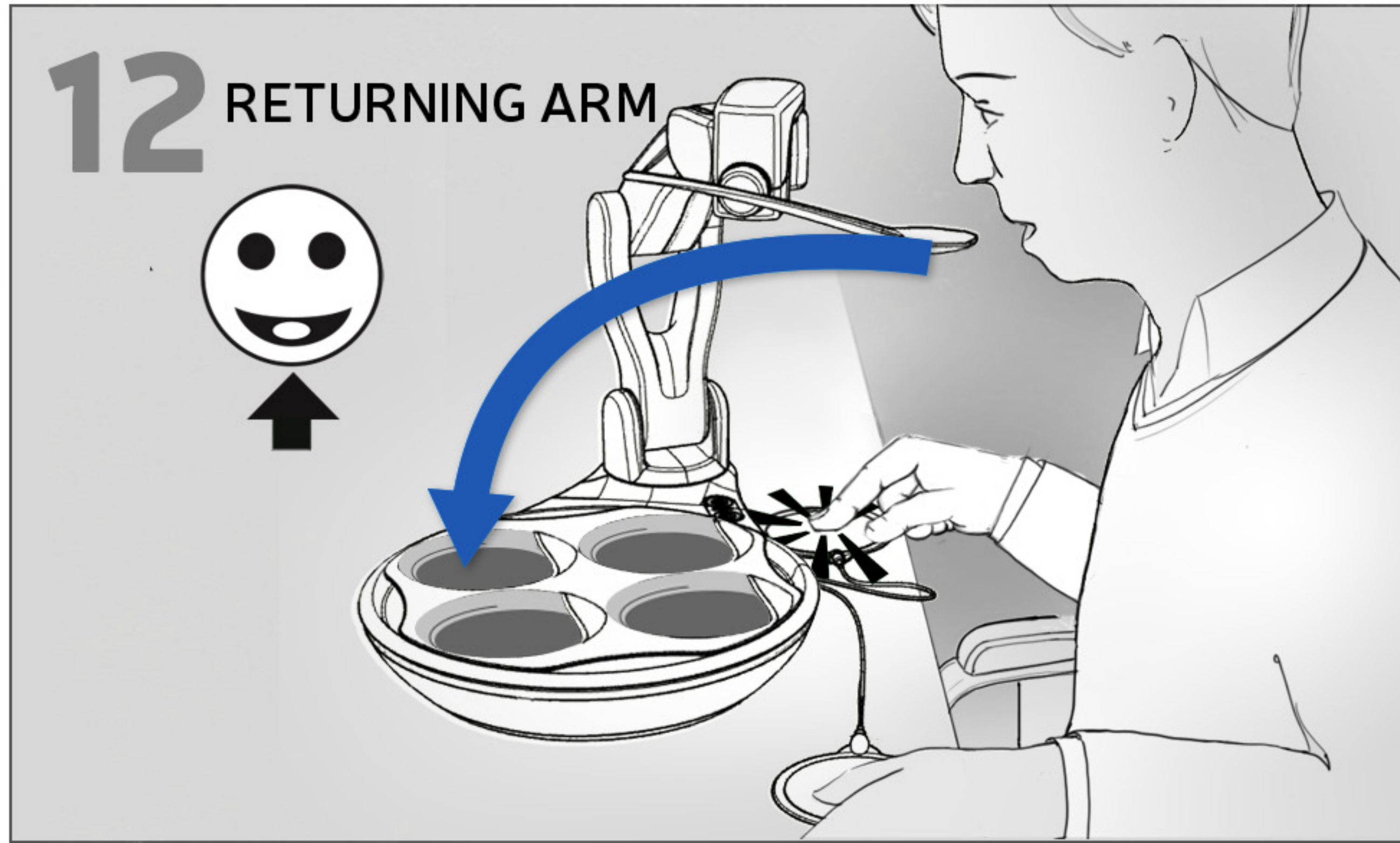


Press the deliver switch to return the spoon to the bowl.

## CLEANING

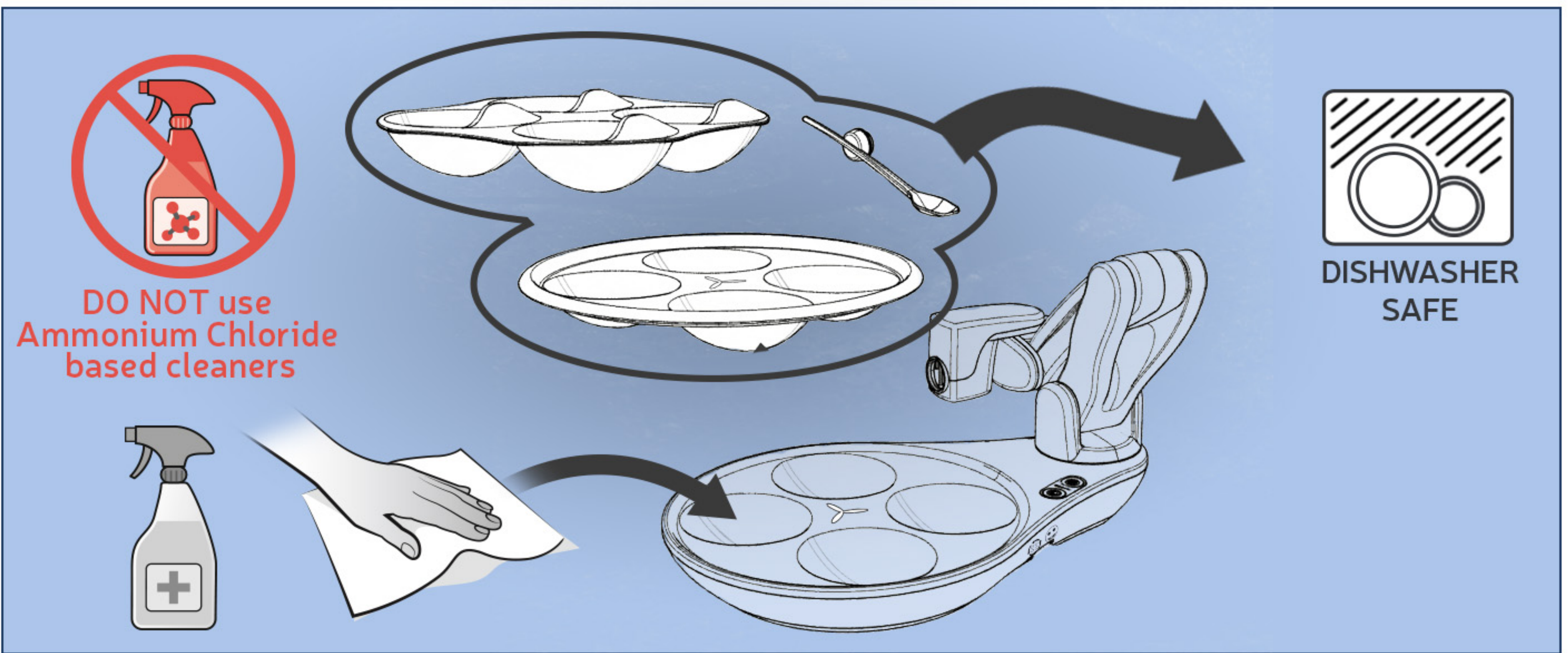


Clean/sanitize the plate, placement and utensils in a sink or dishwasher (top rack). Clean/disinfect Obi's base and arm with a compatible solution. See the Instructions for Use for more information.

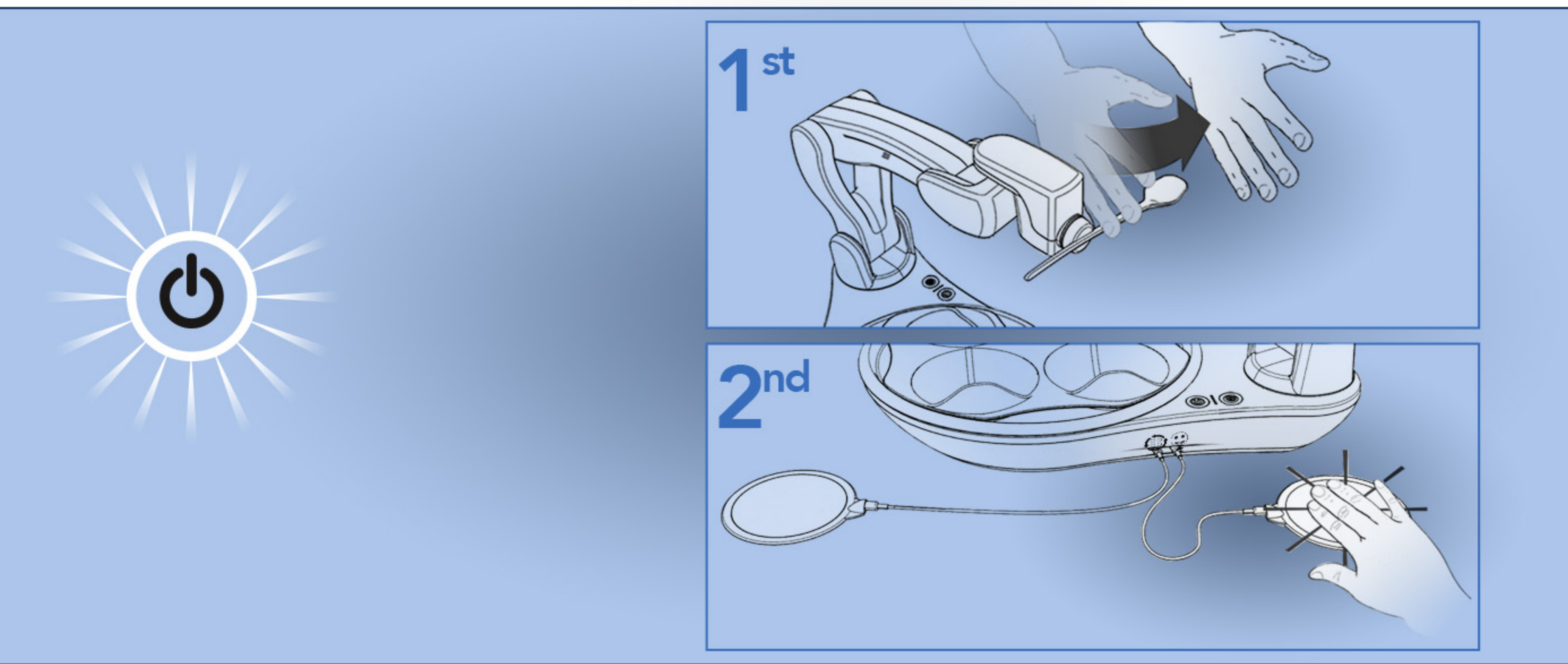

If the power button flashes, please remove the obstructing object. Once removed, press either switch and the arm will resume its motion.

## COLLISION

## CHARGING

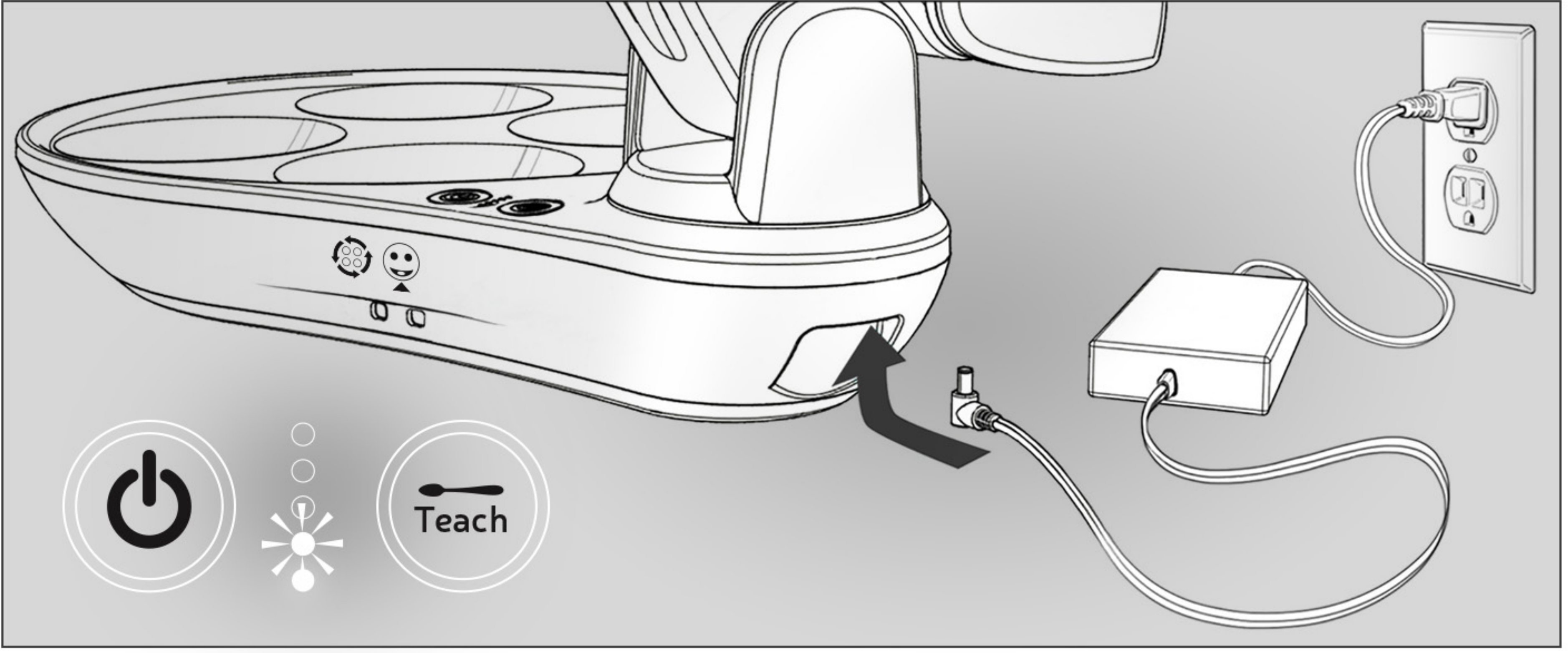

Connect the charging cable. All battery indicator lights will illuminate when the device is fully charged. When storing Obi for a prolonged period, ensure a 40-60% charge to minimize the risk of a dead battery.



If you experience an issue with Obi, Email: [service@meetObi.com](mailto:service@meetObi.com) or Call: 1-844-HelpObi.

## SERVICE