



Key Information for Clinicians & ATPs



INTENDED USE

Obi is intended to restore functional eating performance for individuals living with severe bilateral upper extremity motor impairment.




INDICATIONS FOR USE

5+

years of age

(or younger, with professional approval)

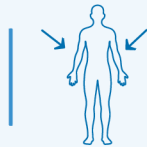

Severe neurological, neuromuscular, or musculoskeletal impairment of the upper extremities



Must have cognitive ability to understand cause and effect

COMMON CONDITIONS

Amputees, ALS, Arthrogryposis (AMC), Cerebral Palsy, Congenital Limb Differences, Developmental Disabilities, Muscular Dystrophy, Multiple Sclerosis, Parkinson's Disease, Rett Syndrome, Spinal Cord Injury, and Spinal Muscular Atrophy



RELATIVE CONTRAINDICATIONS / PRECAUTIONS

A patient should **NOT** use Obi if any of the following conditions are present and of a severity that results in an unacceptable compromise to the benefit/risk ratio provided by the device. Where appropriate, special attention should be given to whether the following attributes pose an unacceptable risk to unsafe feeding or eating.



Cognitive / Behavioral Impairment



Dietary Restrictions



Dysphagia



Open Skin & Mouth Sores



Excessive Movement



Insufficient Head Control



Insufficient Movement



Reclined Positions



SWITCH COMPATIBILITY

Obi is controlled by any part of the body that can activate a switch. Obi is compatible with most switches that utilize a 3.5mm headphone jack adapter.



● = Switch Placement Areas

© All Rights Reserved by DESIN LLC, 2026.

WE'RE HERE TO HELP!

Our team of ATPs is here to assist you with the process!

www.MeetObi.com | Info@MeetObi.com | 904-203-8679



This infographic is not a substitute for proper clinical evaluation by an appropriate healthcare professional.

Always follow the prescribing healthcare professional's instructions.